## HEMOCLIPS

### Features

Use polymers or insulating Titanium materials

### Indication

They are designed to block blood vessels (arteries and veins) during open surgery or laparoscopy, which aims at stopping the bleeding

### Specification

No interference with MRI, CT, X-RAY  
No accessories  
use it with applier  
Sterile: EO Gas

### Instruction

After removing the product from the packaging, we select a suitable applier. We slowly remove the clips from the base so that the pressure is not inserted in the applier's handle to prevent any damage to the applier tab. after removal of the clips, we fixed it on the desired vessel and using the applier

### Warning

Keep dry  
Use only once  
After using several clips, the rest of the clips are discarded  
Don't use it when the package has been opened or damaged

Code No.	Size	Measurement of vessels	Color	Material
15301	S	2mm-6mm	Blue	Titanium
15302	ML	3mm-10mm	Green	Polymer
15303	L	5mm-13mm	Purple	Polymer
15304	XL	7mm-16mm	Gold (brown)	Polymer

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## Suction/Irrigation Sets

### Description

The single-use instrument is convenient, leak-free, and without assembling needs.

Suction button (white), allows simple, safe tissue/clot/secretions flushing.

Irrigation button (blue), allows saline to pass into the surgical site.

Distal tip shaft holes prevent tissue damage due to inadvertent aspiration

Clear ergonomic handpiece for unobstructed visualization

Ergonomic hand-piece ensures surgeon comfort and control

Clear coded trumpet valves and tubing ensure function

identification dual spikes options for Increased flexibility

### Warning

Keep dry

Use only once

Don't use it when the package has been opened or damaged

Sterile EO Gas

Code No.	Product description	Packaging
3361	Suction irrigation set 3mm/28mm	14 Packs/Box
3362	Suction irrigation set 5mm/33mm	14 Packs/Box
3363	Suction irrigation set 5mm/45mm	14 Packs/Box

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## Incision retraction and protection

### Description

AJP Retractors are used in all types of surgery for opening incisional tissues for better surgeon vision protecting incisions from skin contaminations

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## Camera Cover (Laparoscopic Cover)

### Description

To cover and maintain sterility, all types of endoscopic cameras, including: laparoscopic, arthroscopic, cystoscopic, etc., which cover the camera up to the lens junction and are fixed on the lens

Its length is 250 cm and in addition it can be used for all kinds of cables, tubes and probes in surgery.

### Warning

Keep dry

Use only once

Don't use when package has been opened or damaged

Sterile EO Gas

Code No.	Product	Dimension w×l	Packaging
1170	Endoscope cover	15×250cm	100 Packs/Box
1171	Clips Endoscope camera cover	15×250cm	100 Packs/Box
1174	Echo probe cover	2×80cm	100 Packs/Box
1175	Probe cover	10×80cm	100 Packs/Box

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